# BÉABA®

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**EN**

## WELCOME

Thank you for choosing our Béaba hands-free electric breast pump to help you with breastfeeding. Breast milk is the most natural food for your baby which is why Béaba has designed an efficient and comfortable breast pump to help breastfeeding mothers on this great adventure. Before starting, please read these instructions carefully to ensure that you use our product under the very best conditions and keep it for as long as possible.

## WARNINGS AND SAFETY PRECAUTIONS

Read and keep these instructions. Please take account of all the warnings.

- Only use the Béaba electric breast pump for the purpose described in this manual.
- Do not use the electric breast pump while driving, sleeping or if you are feeling excessively drowsy.
- This breast pump is intended for repeated use by one sole person.
- This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved
- Wash all the parts that come into contact with the breast and breast milk before using for the first time and after each use. Wash your hands carefully handling the parts and the breast pump to avoid contamination. Refer to the “Cleaning” section for further information.
- Supervision is required when the Béaba electric breast pump is used near children.
- Children must not play with the appliance.
- The appliance must not be cleaned or maintained by children without supervision
- Do not use the electric breast pump or one of its components if they are damaged.
- Do not place the electric breast pump near appliances that generate a strong electromagnetic field.
- It is advised to consult an approved healthcare professional before using this hands-free breast pump if you have an active implanted device that might be affected by a magnetic field (for example, a pacemaker or an implantable cardioverter-defibrillator).
- If expressing causes discomfort or pain, stop the appliance and place your finger between the breast and the nipple shield to separate them. Then, remove the nipple shield from the breast.
- Expressing your milk should not cause discomfort or pain; if you are unable to express your milk after 5 minutes of use, stop your session and try again a little later. **Consult a healthcare professional if you have a doubt.**

### Warnings to avoid the transmission of infectious diseases:

- If you are a mother and infected with hepatitis B, hepatitis C or HIV (human immunodeficiency virus), expressing breast milk will not reduce or eliminate the risk of transmitting the virus to your baby through breast milk.

### Precautions when using the electrical unit/lithium battery

- The lithium battery is built-in and cannot be removed. Do not dismantle.
- Do not expose the electrical unit to direct sunlight or heat.
- Recharge the breast pump once a month when not used regularly.
- Repairs must be carried out by Beaba: contact our customer service.
- Do not leave the electric breast pump unsupervised when connected to a power source.
- Unplug the mains adapter from the wall socket when the breast pump is not in use, unless the battery is charging. Unplug the mains adapter when charging is complete.
- Keep the electrical unit dry and do not immerse it in water or place it under running water.
- Do not pick up an electrical appliance that has fallen into water. Unplug it immediately.

- This breast pump is protected against overheating: when the normal operating temperatures are exceeded, the pump unit triggers an alarm, switches off and stops charging.
- This appliance must only be supplied at safety extra low voltage corresponding to the marking on the appliance.
- 

**THIS APPLIANCE IS NOT A MEDICAL DEVICE.**

## INTENDED USE

The Béaba electric breast pump is designed to express the breast milk of breastfeeding mothers. The breast pump can be used in the event of difficulties encountered with breastfeeding or when mother and baby are separated. The appliance is designed to be used by just one person.

## OVERVIEW OF PRODUCT & ACCESSORIES

1

x1

2

x1

3

x1

4

x1

5

x1

6

x1

7

21mm x1

8

x1

9

x1

- 1 : Pump motor base with 180 ml collection bottle (pre-assembled with 1 25 mm nipple shield, 1 pump body, 1 silicone membrane (diaphragm), 1 backflow valve)
- 2 : Instructions
- 3 : USB-C cable
- 4 : 1 bottle lid
- 5 : 1 bra extension strap
- 6 : Cotton storage bag
- 7 : 1 21 mm nipple shield
- 8 : 1 additional silicone membrane (diaphragm)
- 9 : 1 additional backflow valve

## FUNCTIONS

This breast pump offers the following functions and characteristics:

- Milk collection kit with silicone nipple shields for gentle pressure, vacuum and massage to stimulate lactation while expressing milk.
- Easy assembly and cleaning.
- Expressing in three phases that imitates baby's natural suckling.
  - "Stimulation" mode: effectively massages the nipple and areola to stimulate the ejection reflex. 9 adjustable suction power levels.
  - "Express" mode: imitates baby's suckling rhythm. 9 adjustable power levels.
  - "2-in-1" mode: combines stimulation and expressing, to optimise the milk ejection reflex.
- Portable breast pump: takes up very little space. Works with the built-in rechargeable battery or via USB to guarantee complete flexibility whether at home or when travelling.
- Hands-free breast pump: fits directly into your bra and leaves your hands free for your other activities.
- Backflow design: silicone membrane acting as a barrier to prevent the backflow of breast milk.
- Low noise level for a quiet and comfortable experience.
- Rechargeable with a mains adapter. Built-in rechargeable lithium battery.

## SELECTING THE CORRECT nipple shield SIZE

- 1) Measure the diameter of the base of your nipple (in mm).
- 2) Add 4 mm to this measurement, this is the size of your nipple shield. A space of about 4 mm between your nipple and the nipple shield allows the nipple to expand while pumping your milk. For example, if the base of your nipple measures 21 mm in diameter, the size of your nipple shield should be 25 mm.




N.B.: your breast pump is supplied with a 25 mm nipple shield and a 21mm flange insert (which are the most commonly used sizes). However, if necessary, you can order 17 mm flange inserts from our customer service.

If you are uncertain about the size of the nipple shield you require, print our measurement guide by scanning the QR code below.

How to know whether the nipple shield is the right size?

- Before pumping your milk, it should be possible to insert your nipple with very little or no areola in the tunnel.
- While expressing your milk, your nipple should be centred in the middle and be entered and removed from the tunnel without rubbing, redness or pain.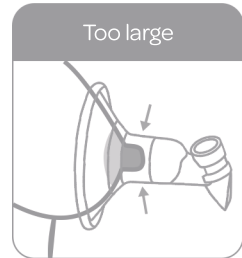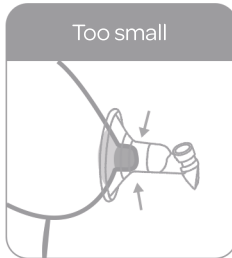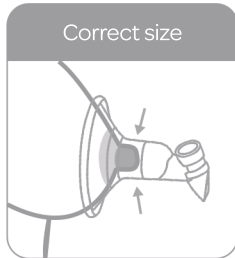


## CLEANING YOUR BREAST PUMP

### Recommendations:

- **When using for the first time**, dismantle, wash and sterilise all the parts of the breast pump that come into contact with the breasts or breast milk. **After each use**, dismantle and wash all the parts of the breast pump that come into contact with the breasts and the breast milk.
- Dismantle and clean the parts that come into contact with the breasts and the breast milk immediately after use to avoid the build-up of milk residue that might favour the growth of bacteria.
- Take care when cleaning the silicone cushion (nipple shield, diaphragm and valve). Your breast pump will not work correctly if they are damaged.
- Always wash your hands before touching the breast pump and its parts. Avoid touching the internal parts of the milk collection kit that come into contact with breast milk.
- The electrical unit must **never** be exposed to or immersed in water or other liquids to clean it. Never wash and never sterilise the pump's motor, this part does not come into contact with breast milk. The electrical unit's motor contains electrical components that must remain dry.

A clean, damp cloth can be used to clean the pump's motor. Any other cleaning method is considered to be incorrect use and will cancel the warranty.

- The appliance **MUST NEVER BE USED** without a diaphragm: risk of fluids entering the appliance's electrical part.
- **NEVER USE** the product without the backflow valve: suction will be irregular and the appliance will be inefficient.
- Always check all plastic or silicone parts for wear.
- Replace used or damaged parts immediately because they will affect the operation of your breast pump.

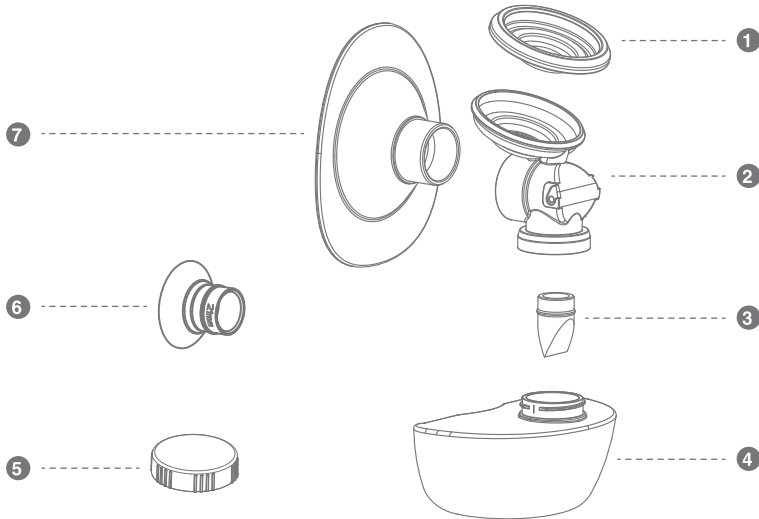Washable components:

- ① : Diaphragm (silicone)
- ② : Pump body (PP: polypropylene)
- ③ : Backflow valve (silicone)
- ④ : 180 ml bottle (Tritan®)
- ⑤ : Bottle lid (PP: polypropylene)
- ⑥ : 21 mm flange insert (silicone)
- ⑦ : 25 mm nipple shield (silicone)

Always handle these elements gently when cleaning. Your breast pump will not work correctly if they are damaged.

**Cleaning the washable components (before using for the first time and immediately after use)**

- 1) Rinse the elements in cold, clear water (about +20°C / +68°F).
- 2) Wash the elements in lukewarm water (about +30°C / +86°F) with a fragrance-free, liquid washing-up product for babies.
- 3) Rinse in cold, clear water (about +20°C / +68°F) for 10 to 15 seconds.
- 4) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 5) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

OR

- 1) Place the elements on the top rack of the dishwasher.
- 2) Make sure that they are completely dry. If necessary, place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

**Sterilising the washable components before using for the first time**

- 1a) Boil in water for a maximum of five minutes.
  - a. During sterilisation, use a teaspoon of citric acid or a little vinegar in 200 ml of water to descale and degrease the pump's parts. Limescale and grease will affect the vacuum and the transfer of milk to the collection kit.
  - b. Boiling or exposing to heat too often may can wear of plastic and silicone parts, which will affect the operation of your breast pump.

OR

- 1b) Use steam or cold water sterilisation methods. We do not recommend sterilising in a microwave.
- 2) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.


**Cleaning the electrical unit:**

Disinfect the electrical unit with disinfectant wipes. Wipe the unit with a clean, damp cloth or paper towel before use. Never place the electrical unit under water and never immerse it. Do not use solvents or abrasive products to clean the electrical unit.

**Scan the QR code below to watch our explanatory video:**


## CHARGING YOUR BREAST PUMP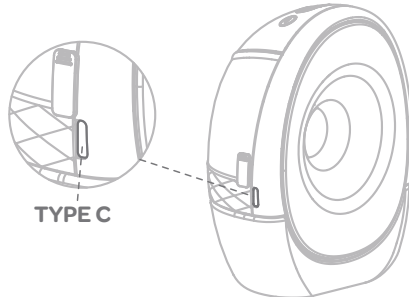


- When the battery is low during operation, the battery light flashes.
- During charging, the battery light comes on and the breast pump cannot be switched on. Use the charging cable supplied to connect the breast pump motor to a USB power source (5V 2A), such as your telephone charger.
- The battery lights all light up when completely charged. It takes about 1.5 hours to charge the battery completely. It can then be used for about two hours with standard settings.
- Only use the charging cable supplied with the portable breast pump. Do not use the portable breast pump while charging.
- Advice: Charge immediately after completing your milk expressing session if the battery is low.

## PREPARING THE BREAST PUMP

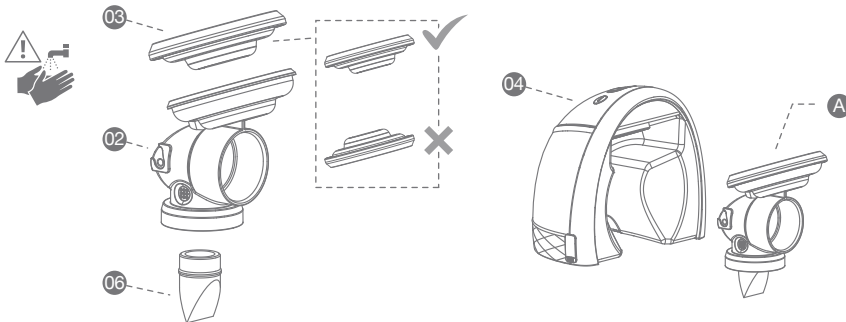
- 1) Make sure that all the parts are dry before assembling them (after the parts have been cleaned and sterilised).


- 2) Attach the diaphragm **03** to the pump body **02**.
- 3) Attach the valve **06** to the pump body **02**.
- 4) Push these elements that have been correctly assembled **A** into the electrical unit's motor **04**. Make sure that it is correctly in place and that the diaphragm can be pressed.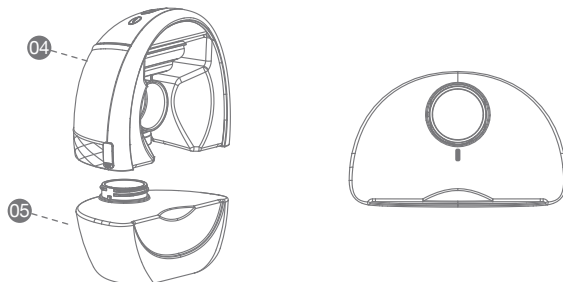

- 5) Screw the bottle **05** onto the electrical unit **04**. Pay attention to the screwing position.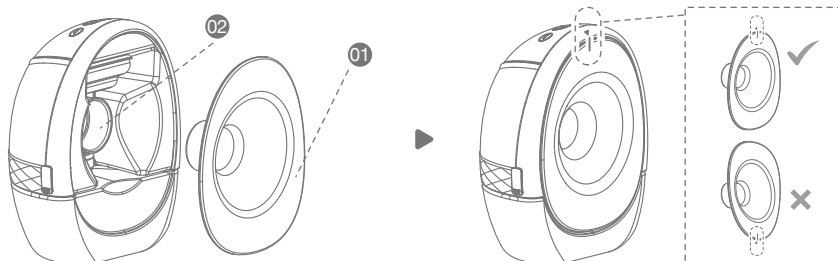


- 6) Push the nipple shield **01** into the pump body **02** and make sure that it is aligned and correctly in place.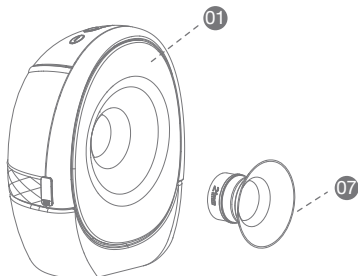


- 7) If the standard 25 mm silicone nipple shield **01** is too big for you, you can use the 21 mm silicone insert flange **07** (or 17 mm to be ordered from our customer service) to place in the centre of the nipple shield.

## INSTALLING ON YOUR CHEST

The correct alignment of the nipple in the nipple shield is essential for comfort, expressing and good performance. Your nipple must be positioned in the tunnel of the nipple shield correctly before starting expressing to avoid any discomfort.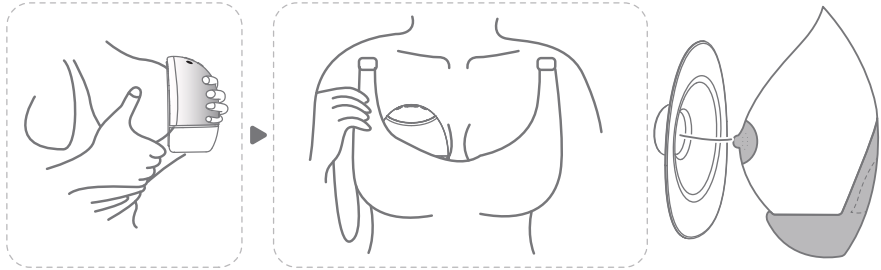

## FINDING THE RIGHT COMPRESSION LEVEL

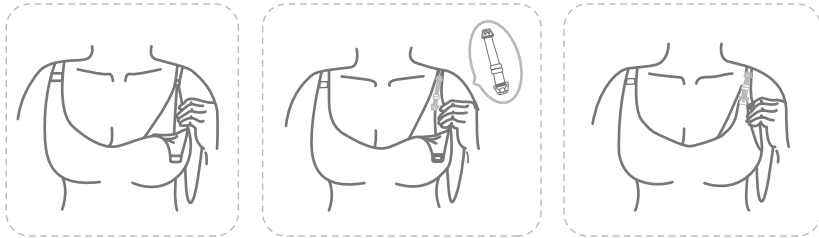
This hands-free breast pump is designed to be installed in your standard nursing bra and can be used either in a seated or standing position.

Obtaining the correct compression level can influence the volume of milk obtained. The breast pump must be pressed firmly against your breast, without any space between the breast pump and your breast.

Every bra offers a different level of support so be sure to adjust your bra strap correctly before each session.

If you use a particularly supportive and rigid bra, the bra strap extension included can be used to create more space. For this, fasten the extension strap to the hooks at each end between the strap and your bra's cup. Then adjust the length for greater comfort.
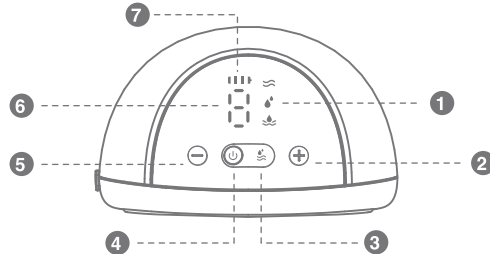
## STARTING THE MILK EXPRESSING SESSION

- 1 Express mode indicator (stimulation , express , 2-in-1 mode )
- 2 Increase power
- 3 Change express mode
- 4 On/Off (Long press to switch on/off, short press to pause/resume)
- 5 Reduce power
- 6 Power indicator
- 7 Battery indicator.

### 1- Start your breast pump

- a. Press and hold the “ON/OFF” button. 4 to switch on the breast pump, the LED screen comes on and switches to standby. Another short press on this button and the breast pump starts working.
- b. During operation, press the «ON/OFF» button. 4 briefly to pause or resume the session if you need to interrupt it for any reason. (When the pause time exceeds 5 minutes, the pump switches off automatically)

### 2- Select express mode

- a. Press the «ON/OFF» button. 4 briefly in standby mode, the breast pump will start automatically in «Stimulation» mode  at the same power level as the last session. When using for the first time, the default setting is to start in Stimulation mode at level 1.
- b. The breast pump will switch automatically to “Express” mode  after 2 minutes. You can also press the “Change expressing mode” button. 3 to switch to “Express” or “2-in-1” mode  if your milk starts to flow sooner.
- c. The 2-in-1 mode includes 5 Stimulation cycles + 5 Express cycles in a loop.

### 3- Adjusting the power


- a. The breast pump has 9 power levels allowing you to control what feels more comfortable and works best in Stimulation, Express and 2-in-1 modes.
- b. Once you can feel the suction on your nipple, use the “+” or “-” buttons to adjust the suction power and find your favourite setting in all the modes.
- c. “Memory” function: the breast pump saves the power setting for each mode when in use.

### 4- Stopping the expressing session

- a. Press and hold the “ON/OFF” 4 button to stop the session and switch off the

- appliance. The breast pump will switch off automatically after 30 minutes of operation.
- Unfasten your bra and use your finger to detach the nipple shield to avoid damaging the nipple.
  - Remove the breast pump from your bra.

#### 5- Storing your milk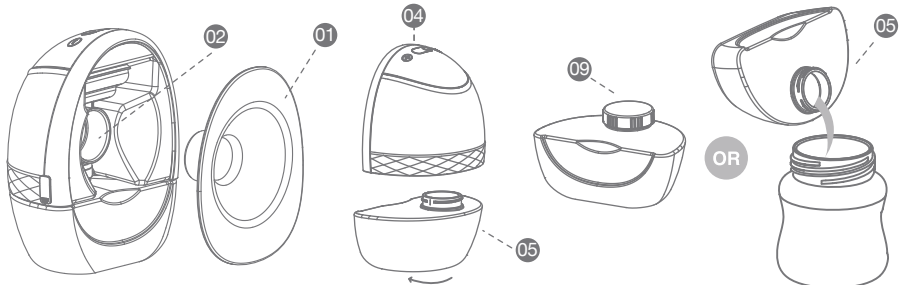

a. After completing your session, remove the nipple shield **01** and unscrew the bottle **05** to remove it from the electrical unit **04**. You can store your expressed milk in the bottle using the lid **09** supplied or pour it into a recipient of your choice suitable for storing milk. The two elements (bottle and lid) supplied are suitable for the fridge and freezer.

#### b. Recommendations for storing your milk in the bottle supplied:

##### **WARNING!**

- Always remove the bottle from the motor unit before storing.
- Always make sure that the bottle and its lid are not cracked or damaged before each use.
- The bottle must not be used to reheat the breast milk. Do not use a microwave or bain-marie to reheat milk.
- Do not defrost the milk in the open but in the fridge.
- Wash your hands carefully before touching the bottle and avoid touching the inside.
- Never refreeze defrosted breast milk.
- Always check the temperature of the milk before serving it to baby.

##### **Recommendations:**



- Remove the bottle from the electrical unit after expressing your milk and close it with the lid supplied.
- Remember to label the bottle with the date and time the milk was expressed and the volume of milk collected.
- Be sure not to fill the bottle to the maximum limit (freezing will increase the volume of milk in the bottle).
- Breast milk can be refrigerated, frozen and defrosted in the BEABA bottle. It is necessary to follow the instructions below:

Breast milk	Storage	Shelf life
Fresh	Room temperature (22°C max)	4 to 6 hours
	Fridge (0 to 4°C)	3 to 5 days
	Freezer (freezer compartment of fridge)	2 weeks
	Freezer (-18°C)	6 to 9 months
Defrosted	Room temperature (22°C max)	2 hours
	Fridge (0 to 4°C)	24 hours


- **Do not refreeze after defrosting.**
- **Please note that these times cannot be combined.**
- These storage and defrosting rules for breast milk are recommendations only.
- **Consult a healthcare professional or a breastfeeding specialist for further information.**

- Defrosting milk:
    - Do not use a microwave or bain-marie to defrost breast milk.
    - Defrost the milk gradually in the fridge.
    - In the event of an emergency, you can defrost breast milk by placing the bottle in cold or lukewarm water (max. 37°C) and using immediately.
    - To reheat the milk, pour the milk into a suitable container and refer to the manufacturer's instructions.
- 6- After the expressing session
- a. Clean all the parts that come into contact with your breast and breast milk (see corresponding section) and inspect for any damage after use.
  - b. Do not store wet or damp parts because mould can develop.

## 7- How to dismantle your breast pump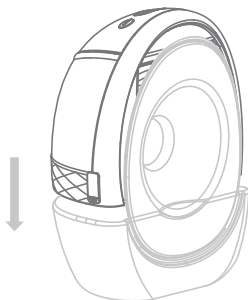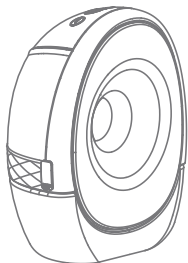

a. Step 1: Pull the bottle with the pump body downwards

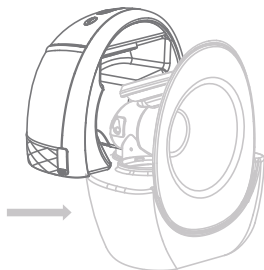b. Step 2: Then pull it towards you

## TROUBLESHOOTING

### *Problem 1: Little or no suction.*

CAUSE	SOLUTION
Check whether the diaphragm is installed upside down or damaged.	Reinstall or replace the diaphragm.
Check whether the connection between the diaphragm and the electrical unit is correctly tightened.	Tighten all the connections.
Check that the nipple shield fits the breast correctly.	Make sure that the nipple shield fits the breast correctly during expressing with the nipple at the centre of the nipple shield's tunnel.

### *Problem 2: Unable to switch on or charge.*

CAUSE	SOLUTION
Check that the charger's output meets requirements and that the charger is connected properly.	Use the correct charger and make sure that it is plugged in.
The lithium battery is depleted.	Recharge the pump when the battery is low.

### *Problem 3: Backflow of milk.*

CAUSE	SOLUTION
The diaphragm is not installed correctly or is damaged.	Stop pumping immediately, switch off the pump and clean it. Do not use it until the diaphragm has been installed correctly or replaced by a new diaphragm.

If your problems with the Béaba breast pump persist or if you have any other questions, please contact our consumer service.

## TECHNICAL DATA

Recommended mains adapter: Input 100-

240V 50/60Hz, Output 5VDC, 2A – 10W

Battery type and capacity: Li-ion 3.7V  
2200mAh

Timer: 30 minutes

Noise level: ≤ 50 dB

Protection class: class III

Operating conditions: temperature 5-40°C,  
relative humidity 10%-90%

This symbol, displayed on the product or on its packaging, indicates that this product complies with the European Directive 2012/19/EU and must not be disposed of with household waste. It must be taken to an appropriate collection point for recycling electrical and electronic equipment. By ensuring that this product is disposed of in an appropriate way, you will be helping to prevent potential harmful consequences for the environment and human health. Recycling materials will help to conserve natural resources.

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**BEABA ASIA LTD**